
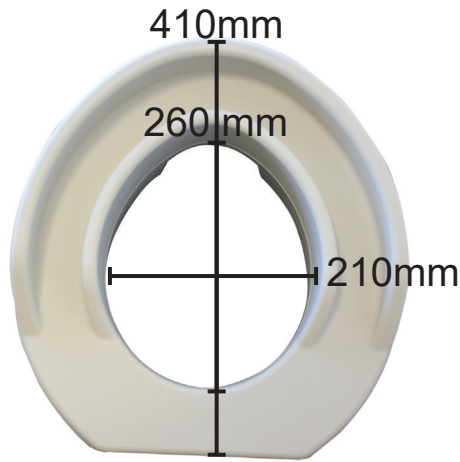
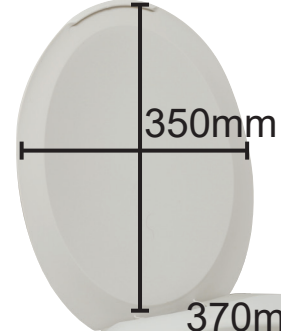
500200n

385mm

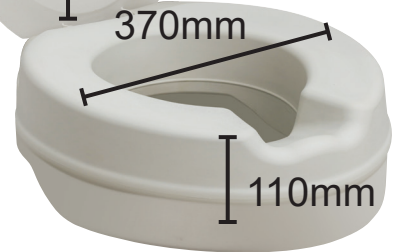
500300n

500250

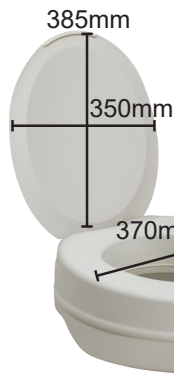2.80 kg

500350

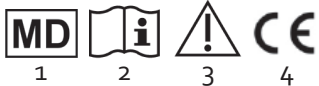
3.20 kg

2 kg

2.5 kg

500200n
500300n
500250
500350

185 kg

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EN-Please read these safety instructions carefully before use, and keep them for future question.

1.Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Do not lean out of the raised seat to avoid falling out of the device.
- ♥ Do not seat or step on the lid.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use outside but only inside and on standard toilet bowls.
- ♥ Use only as a raised toilet seat and in proper way : don't knee or step on the product, don't seat on the lid.
- ♥ ALWAYS observe the weight limit on the labeling of your device.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the product, make sure that it is securely locked and fixed before using.
- ♥ In case you didn't use your raised seat for a long time, let it checked by the technical staff, before using.
- ♥ Always check the stability before using.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.
- ♥ The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. Intended use, indications and contraindications

Intended use:

A raised toilet seat is used to raise the height of a toilet to a comfortable level, making it easier to sit down and get up. Raised toilet seats make it easier to use the toilet independently, maintaining dignity and privacy for anyone with impaired mobility. Raised toilet seats can be used at home or in nursing homes.

Indications

Lack of trunk control - Loss of balance - Difficulty sitting and standing - Stiffness in the hips and/or spine - Weakness in lower extremities - Hip replacement

Contraindications

Don't use in case of:

- Severe cognitive disorder - Severe loss of balance - Sitting impairment

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again. Do not use the product if you see any risk.

- 1-Lift the toilet seat.
- 2-Push the raiser downwards on the toilet bowl until it fits.
- 3-Check that the «lips» of the raiser are covering the toilet bowl edge.
- 4-Check stability before use.
- 5-Remove simply the raiser to clean and restart from point 1 to install.

Rental: this product can be rented under the condition that it was correctly cleaned. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The toilet seat raiser has to be cleaned daily with a non-abrasive cleaner. It can be cleaned with products containing no chlorine, such as Surfanios (Anios)

laboratories), or with soapy water. Do not use any corrosive or abrasive detergents, or pressure cleaners. Sterilisation is possible at 90°C (194°F) and 80 % humidity for 5 minutes.

4. Materials

The seat is made of one piece, the skin is made of PVC and the inside made out of high-resiliency foam.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2-year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Duration of use: 2 years.

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