Aquatec® Pico / Aquatec® Pico 3 in 1

en	Shower stool / Shower- and Toilet Chair User Manual	3
de	Duschhocker / Dusch- und Toilettenstuhl Gebrauchsanweisung	6
fr	Tabouret de douche/Chaise de douche et de toilettes Manuel d'utilisation	9
it	Sgabello da doccia / Sedia da doccia e da toilette Manuale d'uso	12
es	Taburete de ducha / Silla de ducha e inodoro Manual del usuario	14
pt	Banco de duche / Cadeira de duche e sanita Manual de utilização	17
nl	Douchekrak / douche- en toiletstoel Gebruiksaanwijzing	20
da	Badetaburet/bade- og toiletstol Brugsanvisning	23
fi	Suihkujakkara / suihku- ja wc-pyörätuoli Käyttöohje	26
no	Dusjkrakk / dusj- og toalettstol Bruksanvisning	29
sv	Duschpall / dusch-och toalettstol Bruksanvisning	32
et	Dušitool / duši- ja tualetitool Kasutusjuhend	34
lt	Dušo sėdynė / dušo ir tualetu kėdė Naudojimosi instrukcija	37
lv	Dušas taburete / dušas un tualetes krēsls Instrukciju rokasgrāmata	40

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

AQUATEC®

Yes, you can.®

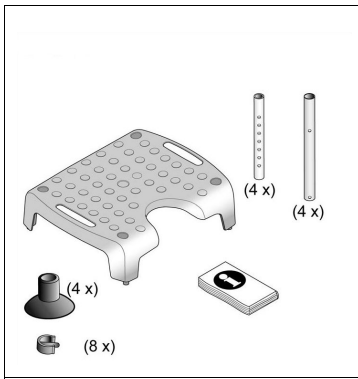
Fig. 1

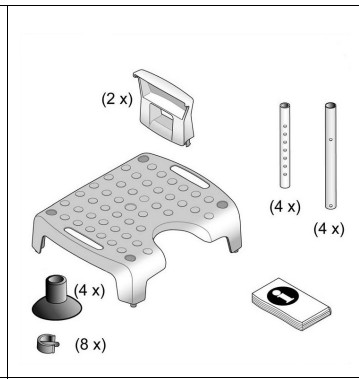
Fig. 2

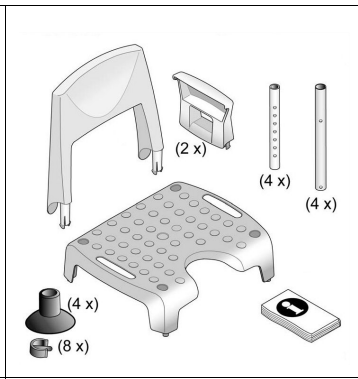
Fig. 3

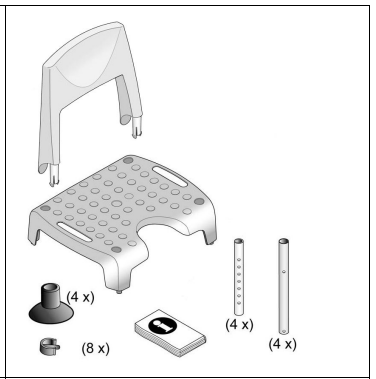
Fig. 4

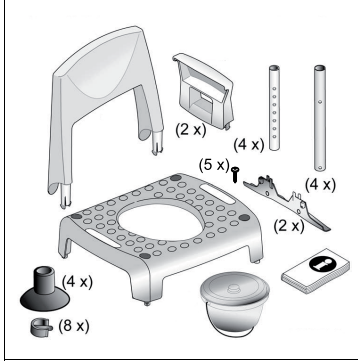
Fig. 5

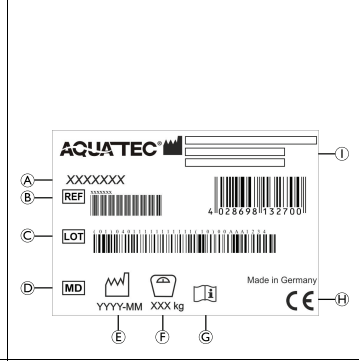
Fig. 6

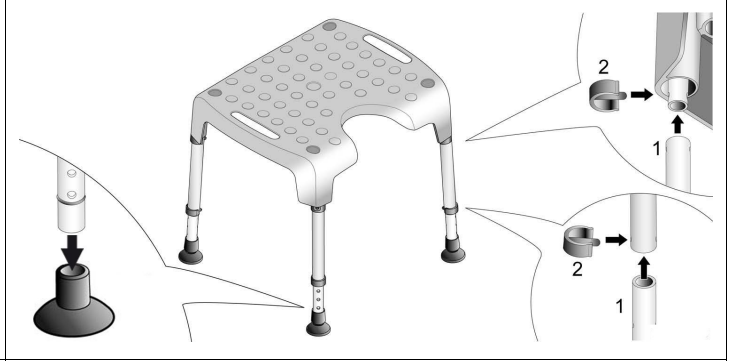
Fig. 7

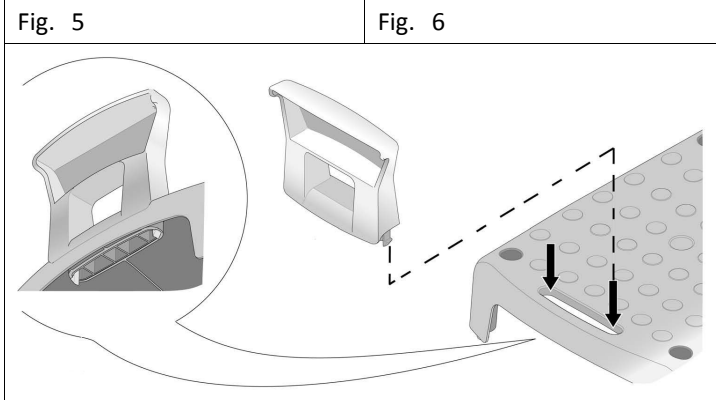
Fig. 8

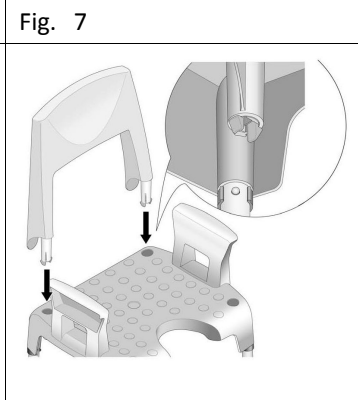
Fig. 9

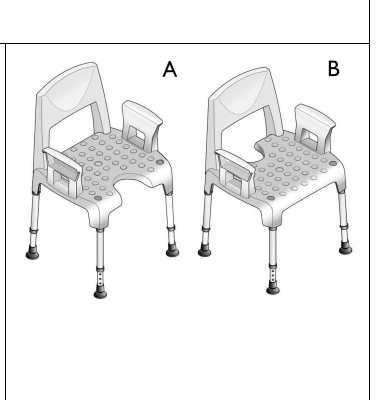
Fig. 10

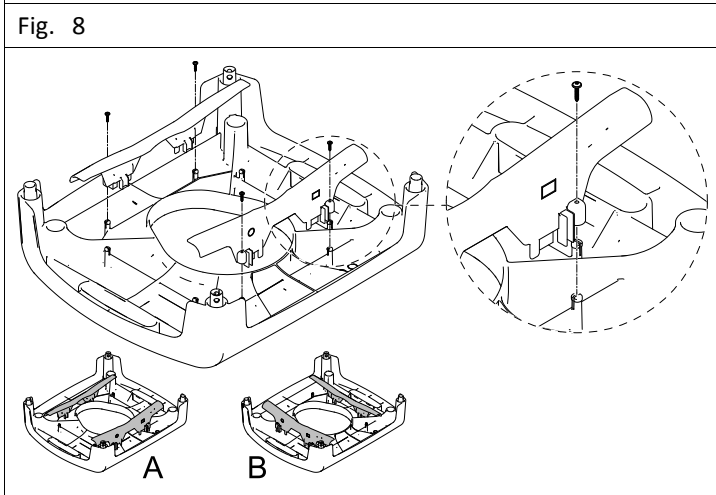
Fig. 11

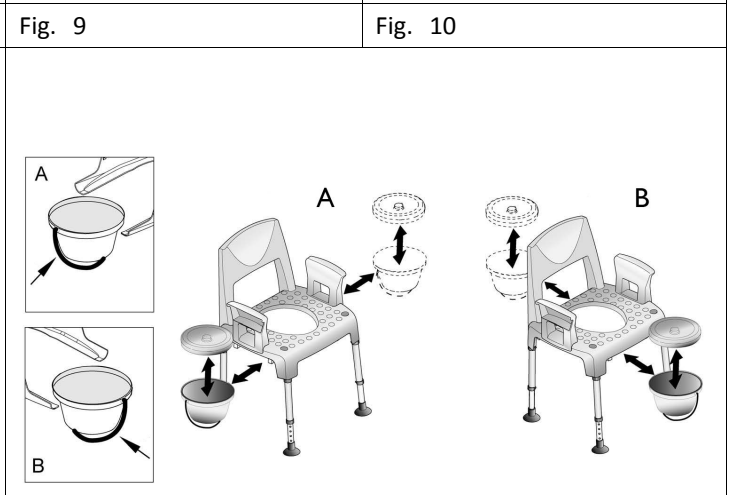
Fig. 12

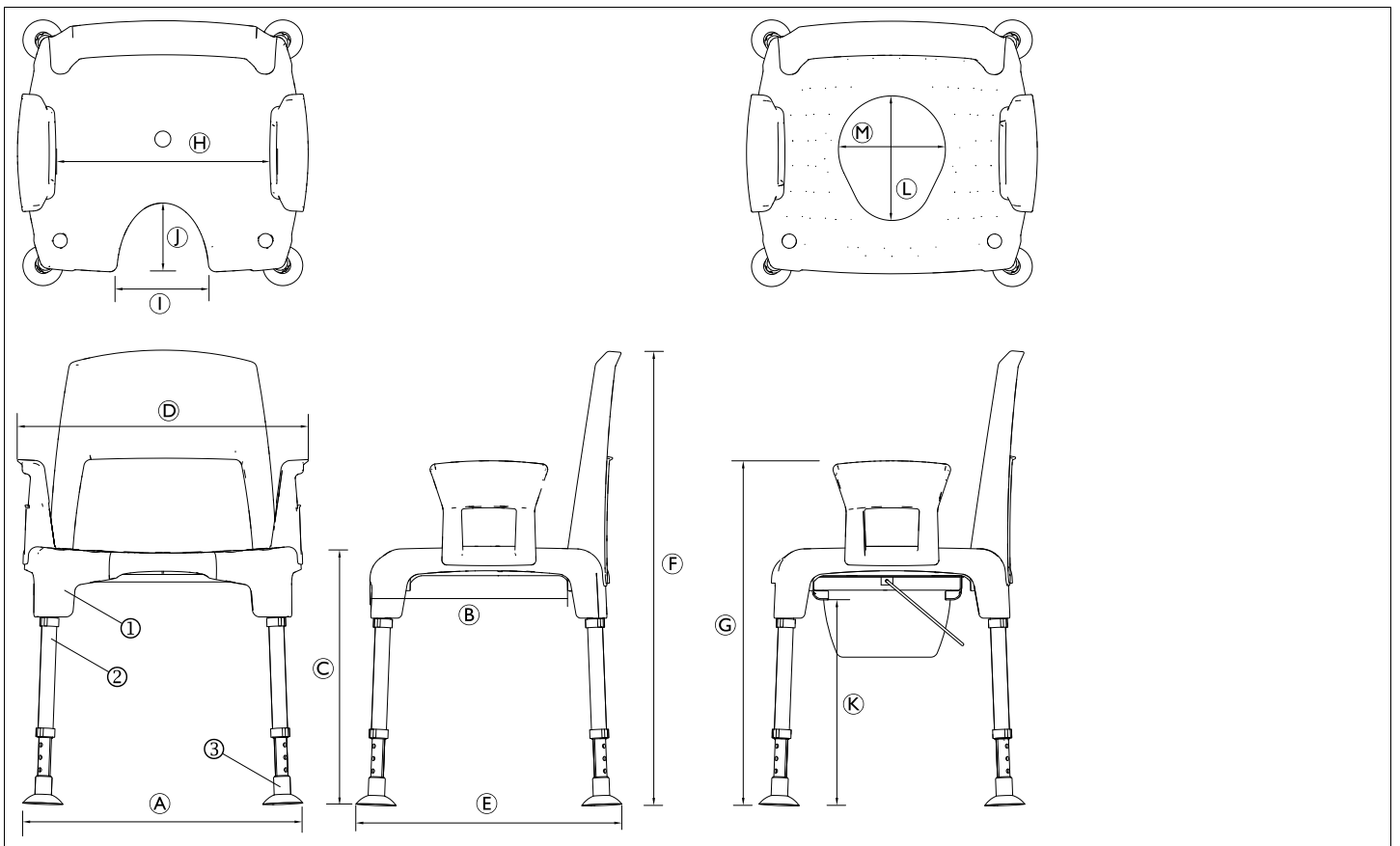
Fig. 13

en

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific standards

The product has been tested and conforms to ISO 17966 (Assistive products for personal hygiene that support users — Requirements and test methods) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.
- Only use accessories intended for this product.
- Do not stand on this product.
- Do not use as a climbing aid.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

6.2 Labels and symbols on the product (Fig. 6)

(A)	Product designation	(F)	Maximum user weight
(B)	Reference number	(G)	Read user manual
(C)	Lot number	(H)	European Conformity
(D)	Medical Device	(I)	Manufacturer
(E)	Production date		

The identification label is attached to the underside of the product.

7 Product description

Aquatec® Pico is a modular plastic shower chair. It is composed of a stool with hygiene recess, with four anti-slip and height-adjustable feet, two removable armrests and one removable backrest.

Aquatec® Pico 3 in 1 is a modular plastic shower- and toilet chair. It is composed of a stool with toilet opening, with four anti-slip and height-adjustable feet, two removable armrests and one removable backrest.

7.1 Scope of delivery

The scope of delivery depends on the version ordered. The possible versions are shown in figures 1 - 5.

Fig.	Version
1	Shower stool Pico
2	Shower stool Pico with armrests
3	Shower stool Pico with armrests and backrest
4	Shower stool Pico with backrest, without armrests
5	Shower- and Toilet chair Pico 3 in 1

8 Intended use

Aquatec® Pico is intended to be used as a shower aid.

Aquatec® Pico 3 in 1 is intended to be used as a shower aid providing a toilet function. The Aquatec® Pico 3 in 1 can alternatively be used over standard toilets for additional support.

Users are any persons with restricted or limited movements (e.g.: legs and/or hips). If used without an assistant, the user must be able to keep his/her own balance.

Indication

Restricted or limited movements (e.g.: legs and/or hips) or having balance issue to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 160 kg.

9 Assembling the product

Installing the feet (Fig. 7)

CAUTION!

Risk of pinching

- Take care during assembly.

1. Push the four anti-slip suction feet completely onto the height-adjustable telescope tubes.
2. Install the four height-adjustable telescope tubes to the legs and secure with clips.
3. Install the four assembled legs to the tube ends on the underside of the seat and secure with clips.


CAUTION!

Risk of injury or damage

- Make sure that all eight clips are securely in place.

All four legs generally should be adjusted to the same height. As an exception, the front legs may be lowered by one position to improve the seating comfort when restricted posture is given due to problems with the coccyx etc.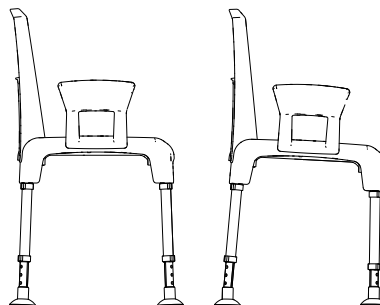

**WARNING!****Risk of tilting**

- Never adjust the rear legs to a lower position than the front legs.
- When lowering the front legs, make sure that they are lowered by maximum one position.

Installing the armrests and/or the backrest

1. Insert the armrests from above into the seat plate until they engage with an audible click (Fig. 8).
2. Insert the backrest from above into the seat plate until it engages with an audible click (Fig. 9, 10).

To remove the armrests/backrest press together the snap-fits on the underside of the seat and pull out the armrests/backrest.

Installing the toilet pan holders (Fig. 11, 12)

The toilet pan holders can be mounted crosswise (A) or lengthwise (B).

Five screws are included in the scope of delivery (Fig. 5) for assembling the two toilet pan holders. Only four screws are required for assembly.

1. Install the toilet pan holders using the delivered screws.

**IMPORTANT!****Risk of damage to the product**

- The plastic parts can crack when the screws are overtightened.
- Tighten the screws by hand only.

2. Slide the toilet pan into the holders under the seat.

The product is disassembled in reverse order to assembly.

10 Cleaning and Disinfection

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

10.1 General Safety Information**CAUTION!****Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**IMPORTANT!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

10.2 Cleaning Intervals**IMPORTANT!**

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

10.3 Cleaning

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

10.4 Disinfection Instructions

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

In Domestic Care

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

In Clinical and Long Term Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

Disinfecting in an autoclave

1. Disinfect the product in an autoclave at max. 85 °C for 3 minutes.

**CAUTION!****Risk of skin burns**

- Allow the product to cool after autoclaving before use.

11 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

12 Technical data (Fig. 13)

Dimensions			
(A)	550 – 560 mm	(H)	428 mm
(B)	445 mm	(I)	181 mm
(C)	430 – 580 mm	(J)	145 mm
(D)	580 mm	(K)	325 – 475 mm
(E)	535 – 545 mm	(L)	250 mm
(F)	825 – 975 mm	(M)	215 mm
(G)	608 – 775 mm		

Weights	
Shower stool	3.2 kg
Shower stool with armrests	4.1 kg
Shower chair with backrest	4.3 kg
Shower chair complete	4.8 kg
Shower and toilet chair with toilet pan (Pico 3 in 1)	5.3 kg
Maximum user weight	160 kg

Materials			
①	PP	③	TPE
②	Aluminum, corrosion-free		

All product components are corrosion resistant and do not contain natural rubber latex.

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Invacare Limited