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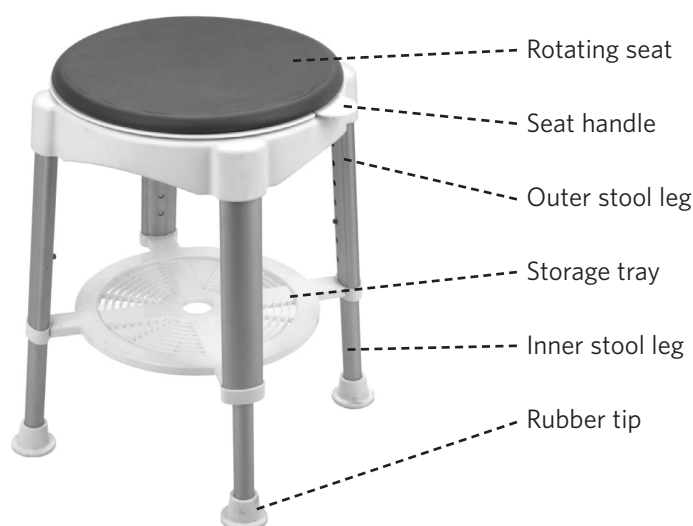
Rotating shower stool with tray

Art-no. 78090 000

Please read the user instructions carefully before using the shower stool, please comply with the instructions and store them in a safe place. If other people should use this aid, they have to be comprehensively informed about the content of this manual.

The shower stool with soft rotating seat and tray helps people with restricted mobility or difficulty in standing caused by lack of lower extremity function following injury or illness. It therefore aids the independence and safety of people while in the shower since it allows them to take a shower sitting down. For people who are sensitive to pressure the soft rotatable seat allows them to sit more comfortably. The rotating seat extends the reach of the user within the shower. Due to its light weight, the shower stool can be removed from the shower cubicle easily when not in use.

⚠ Safety Instructions ⚠

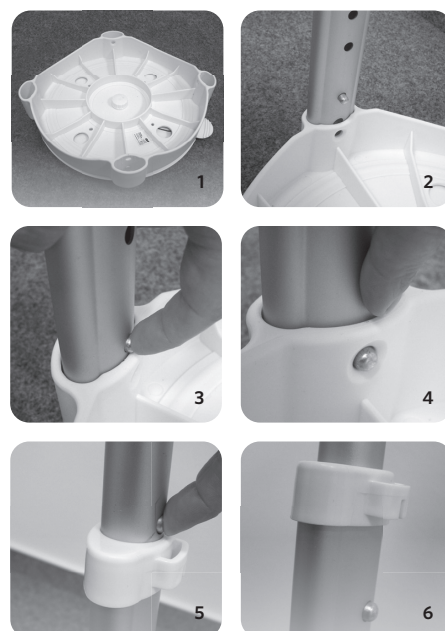
- Only use the shower stool if it is in perfect technical condition!
- Users heavier than 150kg must not use the shower stool!
- After the assembly of the shower stool, please test the stability of the shower stool before use and ensure that all stool legs have been adjusted to the same seat height!
- Please ensure before any use that the shower stool has a stable and secure footing in the shower cabin!
- Do not lean too far forward, backwards or to the side while sitting on the shower stool to prevent the stool from tipping over!
- Do not change the shower stool's height when sitting on it!
- Please observe that in wet shower cabins there is a sliding hazard during sitting down or getting up from the stool - in particular when using cleansing additives or soap!
- Never stand or kneel on the seat!
- Children must not play with the stool or use it unattended!
- There is a risk of getting trapped if you reach into the gap between the swivel seat and the top.
- Do not alter the construction.

Content

1 seat, 4 inner stool legs, 4 outer stool legs, 4 rubber tips, 1 tray.

Assembly

1. Remove contents from carton.
2. Place the seat with the soft swivel seat facing down on a flat surface (1). Insert the outer stool leg into the recess on the surface of the seat (2). The holes for adjusting the height of the outer stool leg should be pointing to the inside of the seat. Press the brass spring pin in (3) and push the outer stool leg into the recess until the spring pin is clearly protruding through the hole in the recess (4). It is securely engaged when a click can also be heard clearly. Proceed in the same way with the other three outer stool legs.
3. Take an inner stool leg and insert it into an outer stool leg by pressing the brass spring pin of the inner leg in (5). It is securely engaged when a click can be heard clearly and the spring pin is protruding from the hole (6). Repeat the process with all other inner stool legs. Make sure that all stool legs are adjusted to the same height!
4. Turn the shower stool right-side-up. Attach the soap tray by placing it between the legs and inserting the cross braces into the receptacles on the white leg collars.

Adjusting the height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. To be sure legs are locked into place, push down on or pull up on the legs.

Rotating seat

To rotate, pull up on handle until seat is free of one of the 4 locked positions and rotate to desired position. Seat can rotate in both directions.

Note: Built into the seat frame are 4 locking positions which will secure the seat in place to allow the user to safely stand up from or sit down on the seat. Make sure the seat is in one of the locked positions before standing or sitting.

Cleaning/Disinfection

The shower seat should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

Material

- Seat: ABS
- Stool-legs: anodised aluminium
- Foot cups: rubber
- Rotating soft-seat: PU foam plastic and polypropylene disk.
- Tray: Polypropylene.

Disposal

Do not dispose of the shower stool in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

Technical data

- Dimensions Seat: Ø 35 cm
- Seat height: 7 changeable stages: 41-58 cm
- Total weight: 3.3 kg
- Foot cups: Ø 5.5 cm
- Maximum load: 150 kg
- Floor space required: 38 x 38 cm

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@ruska.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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Fabrikant: Ludwig Bertram GmbH, Im Torfstich 7, 30916 Isernhagen, Duitsland, www.russka.de