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Saljol comfort back belt


for Saljol Carbon Rollator and Allround Rollator

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Product overview

- 1 Reflective stripes
- 2 Recess for better visibility of the floor in front of you
- 3 Breathable mesh lining
- 4 Fastening rail
- 5 Screws for length adjustment
- 6 Latching nose

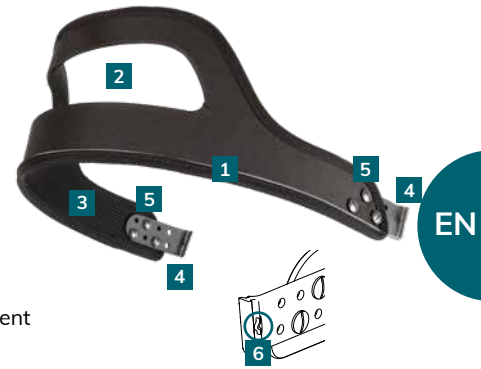EN

Article number: 4058789015261

This back belt fits all Saljol Carbon/Allround Rollator models manufactured as of 12/2023, recognisable by the notch on the holders on the front of the handles.

Assembly/disassembly

Insert the two fastening rails of the back belt into the holders on the front of the handles of your Saljol Carbon/Allround rollator until the latching nose engages with a „click“. When fitting the back belt, always make sure that the reflective strip is on the outside.

Place one finger under the fastening rail as shown. Now pull the fastening rail out vertically upwards with your finger, holding your thumb against the rollator handle.


Caution!


- Make sure that the back belt is fully inserted into the holder at all times. Serious injuries may otherwise occur or damage may be caused to the rollator.
- The maximum user weight must not exceed 120 kg.

Length adjustment of the back belt

The back belt is delivered as standard with the longest possible adjustment. If you would like the belt to be shorter, adjust the fastening rail on one or both sides. To do this, take the belt out of the rollator's holders and place it in front of you. You will need two slotted screwdrivers for this.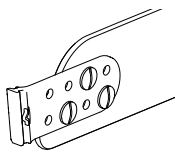


1. Loosen all three screws with a screwdriver. Always hold the second screwdriver against it from the other side. Remove the screws and put them aside.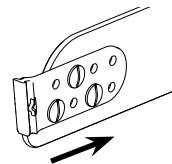


2. Move the fastening rail on the back belt inwards, screw all three screws back into the holes provided in the belt and tighten.

Additional adjustment of the back belt

If the handles of your rollator are set very high, the height of the back belt may therefore not provide a comfortable sitting feeling. In this case, the back belt can also be fitted with the opening recess facing downwards. To do this, take the belt out of the rollator's holders and place it in front of you. You will need two slotted screwdrivers for this. Make sure that the fastening rail is attached to the inside of the belt.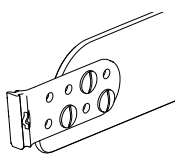

1. Loosen all three screws with a screwdriver. Always hold the second screwdriver against it from the other side. Remove the screws and fastening rail and put them aside.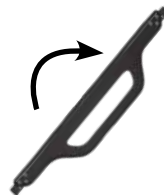

2. Turn the belt 180 degrees, reposition the fastening rail on the inside of the belt to the correct length for you and screw all three screws back into the holes provided in the belt and tighten.

Information correct at time of printing. Technical specifications are subject to change without notice. Technical drawings and photos may differ from reality. ©2024 Saljol GmbH: all rights reserved. Reprints, including excerpts, are only permitted with the permission of Saljol GmbH.

04/2024