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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

### Intended use of the device

The Mobilex “Kakadu” is a medical device intended to be used only as seating support by individuals who require such help during bathing, showering and defecating. It is also a transport option intended to move patients from and to bathrooms and toilets. The device serves a fundamental function and makes the life of both the physically challenged user and the assistant much easier. The device should only be used indoors. Any other use is prohibited.

If you have any questions, please contact your dealer or Mobilex A/S

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

## 2. Security instructions

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During the assembly or during the different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device.
- The device must only be used on a stable surface.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the device is properly balanced and should avoid accidents.
- Do not stand on footrests, and do not sit on the edge of the seat with the legs on the footrests
- When using the device in a stationary position, the brakes MUST be locked.
- Each device must not be loaded with more than specified in the technical data sheet.
- The device is only to be used indoors and on flat surfaces without any obstacles.
- Always maintain a stable sitting position when using this device and users should not be left on the device unattended. Only remove the hole cover when chair is used for defecating.

## 3. Description

### Product description

The “Kakadu” comes with all the standard features of the regular shower chair: excrements bucket, parking brakes, detachable footrests, elevable armrests etc. It is designed for individuals who require seating support when bathing or showering. Made from powder coated steel tubing it is warm to the touch and corrosion resistant. No tools are required for assembly.

The device is supplied in a carton, detached into few elements for easier transportation and assembly. The contents are:

- ✓ 1 main frame with seat and 1 detachable back frame with armrests
- ✓ 2 detachable Footrests
- ✓ 4 detachable wheel assemblies consisting of wheel, brake and vertical assembly tube
- ✓ 4 bolts and 4 nuts for wheel assembly
- ✓ 1 defecation bucket and 1 PU cover for the middle hole of the seat
- ✓ This user manual

#### Assembly description

As shown on the picture above the device can be assembled in quite an instinctive way. We recommend that you follow the below steps:

- Take all the content out of the box and check with the list above that everything is there. If you discover that some parts are missing or damaged, do not continue the assembly process. Contact your dealer right away.
- Put the main frame upside down on the floor. You'll see four holes at the end of the vertical frame tubes.
- Insert the wheel assembly tubes into each frame bottom ending. There are 2 holes (A & B) drilled in both: chair's frame and each wheel's assembly (as shown on the picture). Adjust the wheel's vertical tube to the position, where one of it's holes would match one of the holes in the frame. Due to this solution it is possible to set the seat height of the chair in 3 different heights (in 2,5 cm steps). When required position is reached insert the fixing bolt through all the holes and tie it with the nut from the opposite side of the tube. Always adjust all 4 wheels to the same height. Check if all wheels are locked correctly by pulling them upwards. They should not be able to slide out or shake.
- Now turn the main frame around into it's correct position so it would stand on the 4 wheels. Before further assembly lock all brakes on the 4 wheels, so the device won't move.
- Now slide back frame tubes' endings in to two assembly holes located in the rear, top part of main frame. Tighten the bolts on both sides of the frame, so the back is stable and rigid.
- Put the lid on the bucket. Slide it in to the rails located under the seat. You should do it from the front into the rear direction of the chair. When the bucket is in the place cover the middle hole of the seat with the PU cover.
- Now you can attach the footrests. Put the top of the footrests into the hole located at the top front end of the main frame. The footrest should be directed outwards at that point. Now turn the footrest in until it will lock in it's front position. Repeat the same procedure for second footrest.

#### 4. Function description

- To engage the brakes step down with your toes on the pedals attached to the rear part of each wheel until they are locked. To disengage the brakes pull up the pedals with your toes until they will unlock.
- You should always engage the breaks when you don't plan to move the chair. Never leave chair without breaks engaged when there is someone sitting in it! **Never leave chair on slopes without breaks engaged!**
- You can adjust the length of the footrests to match the height of the user. Release the leg rests length adjustment bolt and slide the footrest tube in to required position. Then tie the bolt so the footrest is fixed and stable.

- Excrements bucket should only be used by persons who have difficulties to use regular toilets. For the bucket to be used remove the middle hole cover. Take the lid off the bucket. When defecation is finished put the lid back on the bucket and pull the bucket out of the chair. Empty the bucket, and wash it before sliding it back under the chair's seat. Put the seat's middle hole cover of the seat back on.
- **Never leave bucket unemptied or unwashed.**
- To unlock and detach the footrest press the locking lever, turn the footrest outwards and then pull it up, of the chair. The elevable foot plates and the above function helps to transfer the user into and out off the chair. It also helps to wash all the parts of user's body during the shower when footrests are removed.
- Never move the chair with footrests detached when there is someone in it. Always put the feet of the user on foot plates when moving the chair
- Armrests can easily be lifted by pulling them up. To move them back into low position simply push them down. This function helps to transfer the user into and out off the chair and to wash the user.
- Never leave chair unattended with armrests lifted when there is someone in it.
- Please note that the footrests and armrests are **not designed to handle the full max. body weight of a user** and should not be loaded like that.

### Transport recommendations (how to lift and carry)

The "Kakadu" commode and shower chair is not a device designed for frequent transportation. If it is necessary to transport the chair, we recommend to disassemble it in the way opposite to the assembly description above. Chair should be packed in the way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

If you need to carry the assembled shower chair, then we recommend to detach the footrests and lift the shower chair on both sides of the frame on the points shown below.


Lifting points for the "Kakadu" commode and shower chair

### Accessories


It is possible to replace the rear 5" small wheels with 24" big wheels with push rims for self propelling. Assembly of this kind of wheels requires also mounting of typical wheelchair brakes. It is all included in the accessory kit (item no. 302017).

### Replacement description: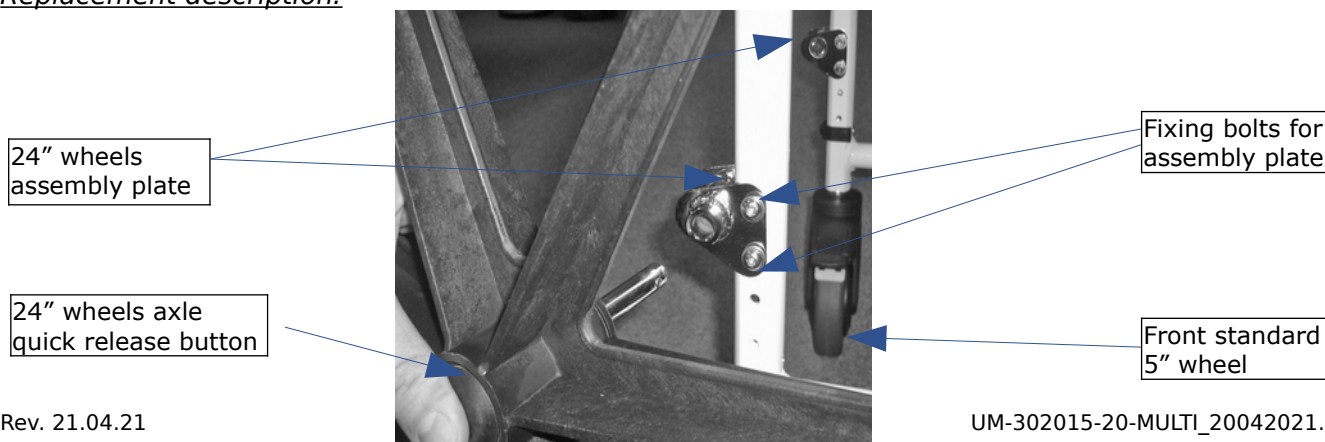

- Disassemble the 2 rear standard wheels in the way opposite to previous assembly instruction.
- Fit the 24" wheels assembly to the rear vertical side frame tube (see above picture).
- There are holes drilled in the tube that allow to adjust the chair to different seat heights. Choose 2 required holes and adjust the assembly plate so it's fixing holes are in line with tube holes.
- Insert 2 fixing bolts through all the holes. Apply washer and nut. Tie the fixing bolts, so the assembly plate is very stable.
- Repeat above steps for the other side of the chair.
- Press the quick release button located in the middle of the big wheel. Insert the wheel into the housing on assembly plate. Release the button.
- Always check if the wheels are attached correctly by trying to pull them out. When mounting the accessory wheel set (302017), make sure that the nut at the end of the axles are well tightened.
- To take away the wheel press the quick release button and pull the wheel out.
- After assembly of big wheels it might be necessary to readjust the height of the front small wheels as it was described previously. The main bottom frame always has to be in position parallel to the floor.

Brake assembly description

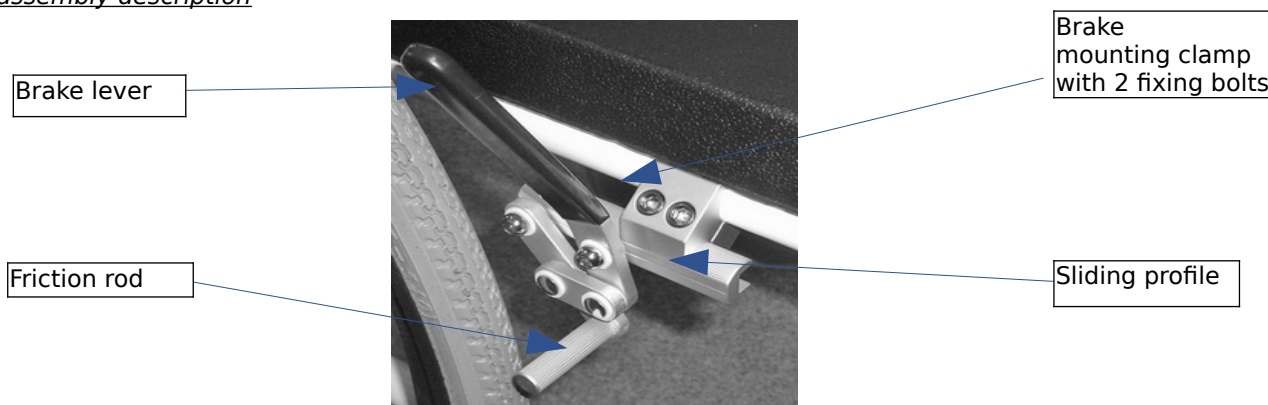
- For the 24" wheels it is necessary to mount regular wheelchair parking brakes as wheels are not equipped with their own brakes.
- The brake assembly consists of mounting clamp, brake mechanism and 2 fixing bolts.
- Insert the sliding profile of the brake into the smaller cut out of mounting clamp.
- Put the mounting clamp on the horizontal side frame tube as it is shown on above picture.
- Move the sliding profile to find the optimum position of the brake, so it is quite easy to apply the brake when necessary, but the friction between tire and the friction rod is strong enough to efficiently hold the wheelchair in stationary position.
- Tie the fixing bolts, so the brake is stable and fixed.
- Repeat those steps for the other side of the chair.
- To apply the brakes simply push the brake lever towards the front of the chair. To release brakes pull the lever towards the back of the chair.

<b>Kakadu</b>	<b>Small 302015</b>	<b>Standard 302018</b>	<b>Large 302019</b>	<b>XL 302020</b>
Seat width	38 cm	45 cm	50 cm	55 cm
Seat depth	38 cm	43 cm	43 cm	43 cm
Seat height adjust.	50 - 55 cm	50 - 55 cm	50 - 55 cm	50 - 55 cm
Seat opening	17 x 33 cm	21,3 x 40,5	21,3 x 40,5	21,3 x 40,5
Pass over height	42 - 47 cm	42 - 47 cm	42 - 47 cm	40 - 45 cm
Pass over width	32 cm	40 cm	45 cm	50 cm
Total width	49 cm	56 cm	61 cm	66 cm
Total depth	85 cm	93 cm	93 cm	93 cm
Total height adjust.	96-101 cm	96-101 cm	96-101 cm	96-101 cm
Max. Load	130 Kg	150 Kg	150 Kg	200 kg
Weight	11,00 Kg	12,90 Kg	13,50 Kg	14,10 kg

## Product label

The product label is located at the bottom of the frame

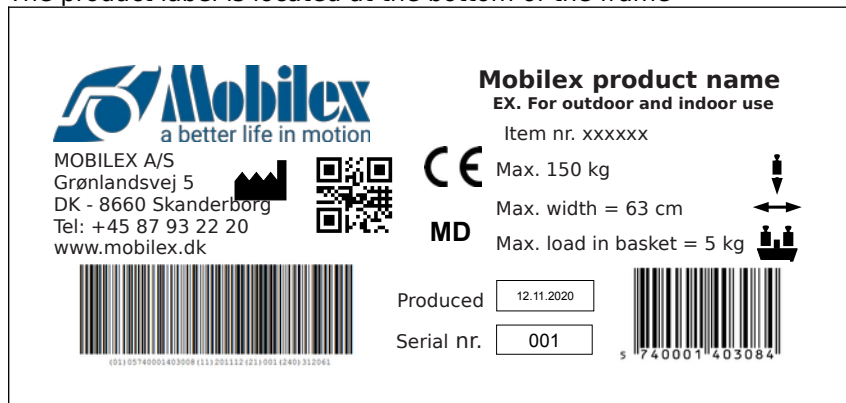Example of the product label (not original)

## 5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

**Recommended service life:** Normally 10 years depending on use.

## Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

### Warning

 If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

### Warning

 When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

### Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

## 6. Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

## 7. Most common problems and solutions

Symptoms	Possible cause	What to do?
The chair is not stable and can't reach horizontal position	The wheels are adjusted to different heights	Adjust all wheels to the same height
The wheels do not turn smooth and easily	The space between wheels and forks is filled up with dirt	Clean the space between wheels and forks of all unintended objects and elements

## 8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,