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# Badestufe stapelbar

Art.-Nr. 78056 100

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# Stackable Bath Step

Item No. 78056 100

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The bath step provides safe support while stepping into and out of the bathtub. Thanks to the choice of material, it is perfectly designed for wet rooms. The bath step can also be used anywhere in the household where an extra step is needed, e.g. to reach items on a high shelf.

### Safety Indications

- People weighing more than 190 kg must not use the step!
- Please check the stability of the step before each use!
- Only use the step when it is technically in top condition!
- Do not make any constructional changes to the step without the authorization of the manufacturer!
- Please note that there is always a danger of slipping when exiting the bath tub – this danger is especially high when bath additives or soap has been used!
- Only use the bath step for its original intention – the step is not a toy!
- Children may not use or play with the step alone due to the danger of falling.

### Scope of delivery

The product contents include a bath step, instructions for use and packaging cardboard. Please check the packaging contents. For noticeable defects or missing parts, please contact your specialized dealer.

### Assembly

The adjustable bath step is delivered ready for use.

#### Use of multiple steps

The bath steps can be stacked. A maximum of 6 steps can be stacked on top of one another. Steps stacked on top of one another do not increase the height by 10 cm but only by 5 cm.

The bath steps can also be connected in order to create a set of steps for easier entry and exit of the bath tub. In order to create stairs the steps are set up so that the plastic edge of one step is inserted into the slot in the next step.

The bath step is delivered with rubber feet in order to prevent the step from slipping away during use. It is not necessary to remove these rubber feet when stacking or connecting the steps.

### Cleaning/Disinfection

This medical product is only intended for contact with intact skin. A mild cleaner should be used to clean the step manually. Disinfection should be completed with DGHM/VAH surface disinfectants. The selection and use of suitable cleaner and disinfectant should correspond with the manufacturer's data.

### Storage

Please store the bath step in a dry, dark and dust-free area.  

### Material

- Bath Step: polypropylene
- Cover: ethylene vinyl acetate Plastic
- Plugs: polypropylene
- Feet: thermoplastic rubber

## Disposal

Please do not dispose of the bath step with normal garbage. Please contact your local waste management company prior to disposal and inform yourself of the correct method of disposal in your region.

## Technical Data\*

- Total Dimensions: 39 x 39/49 x 10 cm (L x W x H)
- Step Surface Dimensions: 36/46 x 35 cm (L x W)
- Max. Load: 190 kg
- Weight: 1.1 kg

\* Small discrepancies are possible

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at [info@russka.de](mailto:info@russka.de) as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, [www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency](http://www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency).

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