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
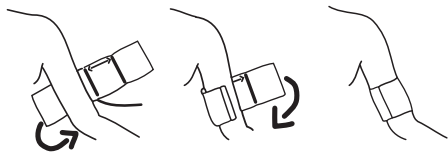
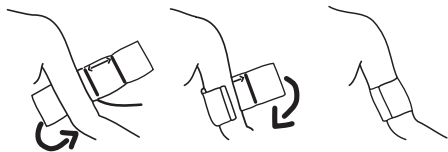
Instruction Manual (*the better choice*)

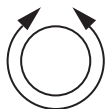1. Apply the cuff to the upper arm/thigh, so that the symbol ϕ is located directly above the artery (brachialis or femoralis).
2. Fit the cuff around the arm/thigh.
3. Make sure that the mark  is arranged in the marked area – otherwise choose a cuff of a different size.
4. Fix the cuff with the Velcro fastener.

Limb circumference

Cuff is applied correctly

Cuff is not applied correctly

Cleaning and disinfection:

Wipe the *the better choice* with a slightly moistened cloth. If necessary the jacket can be washed by using a mild soap or disinfectant solution. Before doing so, please remove the bladder.

Recommended disinfectants:

Cidex, Sporidicin, Microcid, isopropanol (70%), ethanol (70%), Buraton liquid. After disinfection rinse cuff with clear water and let it dry.

Sterilization:

After EO gas sterilization the parts which have been exposed to EO have to be aired thoroughly. All instructions and safety pre-cautions have to be followed strictly. Never autoclave!

Warning! When the blood pressure cuff is not connected to a monitor, the air hose has to be sealed with a plug in order to prevent liquids from flowing in.

Replacement of blood pressure cuffs:

The cuffs have to be replaced if they are damaged, clogged, visibly soiled or when the malfunction detection system of the monitor indicates a whole.

This product is latex free.

Please follow the user documentation of the respective device.

This product complies with the requirements of the directive 93/42/EEC (dated June 14, 1993) for medical products.